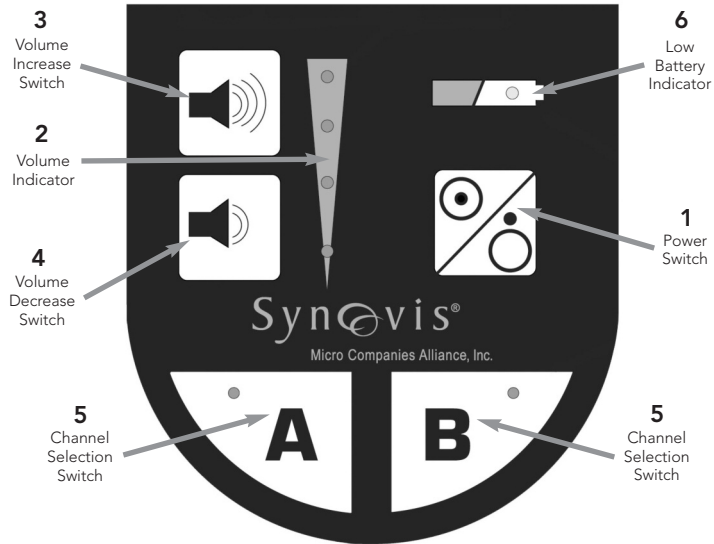
Synovis®  
Micro Companies Alliance, Inc.

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**SYMBOL DEFINITIONS:**

	Consult <i>Instructions for Use</i>
	Made in the U.S.A.
	Content
	CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.
	Type CF Applied Part
	Date of Manufacture
	Serial number
	Catalog number
	Batch code
	Part number
	SSI internal code

**DESCRIPTION:**

The Synovis MCA GEM FLOW COUPLER® System consists of a Flow Coupler and a FLOW COUPLER Monitor. The FLOW COUPLER Monitor is a pulsed Doppler ultrasound system designed for the detection of blood flow in vessels. The FLOW COUPLER includes a 20MHz ultrasonic Doppler transducer (probe) attached to one of the FLOW COUPLER rings, and an external lead. The probe via the external lead connects to the monitor and emits a pulsed ultrasonic signal. A varying audible signal is produced when the probe detects flow.

**INDICATIONS FOR USE:**

The FLOW COUPLER Device is a single use, implantable device that is intended to be used in the end-to-end anastomosis of veins and arteries normally encountered in microsurgical and vascular reconstructive procedures. The FLOW COUPLER Device includes a pair of permanently implanted rings which secure the anastomosis and a removable Doppler probe that is press-fit onto one of the rings. When the FLOW COUPLER Device is used in conjunction with the FLOW COUPLER Monitor, the FLOW COUPLER System is intended to detect blood flow and confirm vessel patency intra-operatively and post-operatively at the anastomotic site. Post-operatively, blood flow can be detected on an as-needed basis for up to 7 days. The FLOW COUPLER Doppler probe is not intended to be a permanent implant and should be removed 3 to 14 days post-operatively.

**CONTRAINDICATIONS:**

The FLOW COUPLER Monitor is not intended specifically to diagnose, monitor or correct a defect of the heart or the central circulatory system.

**WARNINGS:**

If procedures are not followed, injury may occur.

Never sterilize the FLOW COUPLER Monitor with autoclave, ultraviolet, gamma radiation, gas, steam, or heat sterilization techniques. Severe device damage and/or injury may occur.

The Monitor should not contact mucus membranes, blood, or compromised tissue, and is not used in sterile fields.

Not for use in OXYGEN ENRICHED atmospheres.

Do not install batteries backwards, charge, or mix with other battery types. Remove batteries during prolonged storage. Batteries may explode or leak causing injury. Replace all batteries at the same time. Properly dispose of batteries according to local regulations. Do not incinerate.

Monitor not suitable for use in the presence of a flammable anaesthetic mixture with air or with oxygen or nitrous oxide.

Not intended for fetal use.

**CAUTIONS:**

If procedures are not followed, possible equipment or software damage may occur.

The FLOW COUPLER System may be susceptible to picking up interference through the coaxial cable that connects the probe to the Monitor. Do not use in the presence of any high frequency equipment, including high frequency surgical generators.

Only use Monitor with FLOW COUPLER devices.

Only use FLOW COUPLER AC Power Supply.

Only use AA (LR6) alkaline batteries.

Monitor and power supply may be returned to manufacturer for proper disposal.

During the use of all ultrasound devices, the operator should minimize the exposure of ultrasound energy to the patient using the principle of ALARA (As Low As Reasonably Achievable).

## PRODUCT SPECIFICATIONS:

**Monitor Description** (see Figure 1):

1. **Power Switch:** A push-button switch that turns the unit ON. Power ON is indicated by illumination of one of the four green Volume Setting LEDs and a red Channel Selection Switch. When the push-button is depressed a second time the unit is turned OFF.
2. **Volume Indicator:** A series of four green LEDs that indicate the volume of the audible Doppler signal.
3. **Volume Increase Switch:** A push-button switch that will increase the volume of the audible Doppler signal.
4. **Volume Decrease Switch:** A push-button switch that will decrease the volume of the audible Doppler signal.
5. **Channel Selection Switch:** A push button switch that sets the Doppler to a channel. A red LED will be illuminated when the channel is selected.
6. **Low Battery Indicator:** An amber LED which, when illuminated, indicates low battery life. The system will automatically shut OFF when the battery voltage is too low to maintain proper operation of the Monitor.

## Operation:

Transmission Frequency: 20 MHz

Transmission Characteristic: Pulsed transmission, continuous reception

Sensitivity: See FLOW COUPLER Device and System IFU

## Environment:

Ambient Operating Temperature Range: +15° to +40°C.

Ambient Storage Temperature Range: +15° to +40°C.

IPX 0 (Monitor): No special protection

## Power:

Class II External Power Source, A/C to D/C Power Supply

8 AA (LR6) alkaline batteries

Power Requirements: 12 VDC

## Physical:

Dimensions: 8.50in D x 5.00in W x 3.88in H. (216 mm x 127 mm x 99 mm)

Weight: 2.5 lb, (1.13 kg)

## INSTRUCTIONS FOR USE:

1. Carefully unpack your GEM FLOW COUPLER Monitor. Inspect the monitor for damage. If the monitor is damaged, contact the manufacturer for further instructions.
2. Monitor can be powered by batteries (8 AA) or with the external power supply.
  - a. Install the batteries per the instructions in the SPECIAL INSTRUCTIONS section of this manual. Use only AA (LR6) alkaline batteries.
  - b. Connect external FLOW COUPLER AC Power Supply to Monitor. Connect AC Power Supply to supplied Power Cord. Connect Power Cord to a grounded, hospital grade outlet.

**NOTE:** Patient isolation from the AC power is accomplished in the following ways: First, an external medical grade power supply is used to provide DC operating voltage for the FLOW COUPLER System. Both DC power output lines from the power supply are isolated from the AC power in the DC power supply. There is no connection between the "green" safety ground and the Monitor. The implanted probe is further isolated by use of an RF isolation transformer that isolates both signal and ground leads between the Monitor and the probe. The probe's coaxial shield is connected to the Monitor chassis ground through two

safety capacitors. Both the capacitors and the isolation transformer have a 4000VAC dielectric withstand voltage. The final isolation mechanism is the cable insulation and potting of the probe that provides an additional insulation layer between the isolated electrical signals and the patient.

- Place the Monitor on a suitable stand, cart or table outside the sterile field, in close proximity to the physician who will be using the FLOW COUPLER System. The Monitor should not be used in the presence of any high frequency equipment, including high frequency surgical generators. The Monitor has a rating of IPX-0. Keep the Monitor away from all open liquids.

**NOTE:** If the external lead is in close proximity to electrically active conductors, such as an electro-surgery cables or a electronic equipment chassis, signals from the cables or chassis may be picked up by the Monitor and produce undesired audible signals. This interference is easily distinguished from blood flow and is remedied by moving the external lead away from the source of the interference.

- Refer to GEM FLOW COUPLER Device and System Instructions for Use for handling of the FLOW COUPLER Device.
- Transfer the free connector of the external lead (supplied with the FLOW COUPLER device) outside the sterile field. Insert this free connector into either receptacle A or receptacle B on the front panel of the Monitor.
- Turn the Monitor on by depressing the Power Switch.
- Ensure that the Channel Selection Switch (A or B) corresponds with the chosen receptacle.
- Listen for blood flow. Some background noise may be audible.

**NOTE:** If blood flow is not detected with the Monitor post-operatively, rely on clinical indications for patient status.

- Adjust the volume by depressing and holding the Volume Increase Switch or Volume Decrease Switch to the desired level. If a strong audible signal is not identified, irrigate the anastomotic site with saline. During irrigation, an audible signal from the Monitor verifies proper function of the device.

## SPECIAL INSTRUCTIONS:

### Maintenance and Cleaning:

The Monitor requires minimal maintenance. Keep it clean and free of dust. The exterior may be cleaned using the following steps:

- After every use, check the Monitor for any sign of damage or wear.
- Wipe the Monitor with a dry or water-moistened soft cloth.
- Wipe the Monitor with Isopropyl alcohol (70%) moistened soft cloth. Ensure any residual organic material is removed. Do not pour 70% isopropyl alcohol directly on the Monitor. Allow to air dry before use.
- Do not put liquid near the speaker.

### Battery Replacement:

Make sure that the Monitor power is turned OFF by depressing the Power Switch push-button so that no LED's are illuminated.

Unscrew and remove the battery compartment cover of the Monitor, exposing the battery holder within the compartment. Slide the battery holder out of the compartment. Remove both of the battery holder covers exposing the old batteries. Replace old batteries with new, AA (LR6) size alkaline batteries. Observe the battery polarity markings that are molded into the battery holder. Replace the battery holder covers and return battery holder to battery compartment. Replace battery compartment cover and tighten screws. Properly dispose of old batteries according to local regulations.

**NOTE:** Remove batteries during prolonged storage. Do not install backwards, charge, put in fire, or mix with other battery types. May explode or leak causing injury. Replace all batteries at the same time.

## The Following Sections apply to the FLOW COUPLER 20 MHz Doppler Probe

### Acoustic Output:

There are no user controls meant to affect acoustic outputs. All acoustic outputs are below the application specific pre-amendments acoustical output limit of an  $I_{\text{spta},3}$  of 94 mW/cm<sup>2</sup> and a MI of 1.9.

### Explanation of Derivation of Derating Factor

The derated intensity calculations are based on the measured center frequency of the acoustical signal (f, MHz) and the distance from the transducer under test to the hydrophone (z, cm) using the derating factor  $e^{-0.009 fz}$ .

## TRACK 1 SUMMARY

System: 20 MHz FLOW COUPLER Monitor and Probe System  
 Monitor: 20 MHz FLOW COUPLER Monitor

Clinical Application	Mode of Operation						
	B	M	PWD	CWD	Color Doppler	Combined (Specify)	Other* (Specify)
Ophthalmic							
Other (intra-operative and post-operative)			X				
Cardiac							
Clinical Application							

### SYMBOL INDEX

<b>PWD:</b>	Pulsed Wave Doppler
<b>MI:</b>	Mechanical Index
<b>I<sub>SPTA,3</sub>:</b>	Derated spatial-peak, temporal-average intensity (milliwatts per square cm)
<b>I<sub>SPPA,3</sub>:</b>	Derated spatial-peak, pulse-average intensity (watts per square cm)
<b>P<sub>r,3</sub>:</b>	Derated peak rarefactional pressure (megapascals)
<b>W<sub>o</sub>:</b>	Ultrasonic power (milliwatts)
<b>f<sub>c</sub>:</b>	Center frequency (MHz)
<b>z<sub>sp</sub>: (cm)</b>	Axial distance at which the reported parameter is measured (cm)
<b>x<sub>-6</sub>, y<sub>-6</sub>:</b>	Respectively the in-plane (azimuthal) and out-of-plane (elevational) -6 dB dimensions in the x-y plane where z <sub>sp</sub> is found (cm)
<b>PD:</b>	Pulse duration (microseconds)
<b>PRF:</b>	Pulse repetition frequency (Hz)
<b>EDB:</b>	Entrance beam dimensions for the azimuthal and elevational planes (cm)

## ACOUSTIC OUTPUT FORMAT FOR TRACK 1

### Non-Autoscanning Mode

System: 20 MHz FLOW COUPLER Monitor and Probe System  
 Operating Mode: PW Doppler  
 Transducer Model: 20 MHz Doppler Probe  
 Application(s): Other (intra-operative and post-operative)

Acoustic Output		MI	I <sub>SPTA,3</sub> (mW/cm <sup>2</sup> )	I <sub>SPPA,3</sub> (W/cm <sup>2</sup> )	
<b>Global Maximum Value</b>		0.0280	29.4	0.500	
Associated Acoustic Parameter	P <sub>r,3</sub> (MPa)	0.124			
	W <sub>o</sub> (mW)		0.0799	0.0799	
	f <sub>c</sub> (MHz)	20.0	20.0	20.0	
	z <sub>sp</sub> (cm)	0.200	0.200	0.200	
	Beam dimensions	x <sub>-6</sub> (cm)		0.0591	0.0591
		y <sub>-6</sub> (cm)		0.0606	0.0606
	PD (μsec)	0.753		0.753	
	PRF (Hz)	7.81E+4		7.81E+4	
EBD	Az. (cm)		0.05		
	Ele. (cm)		0.05		
Operating Control Conditions	Control 1				
	Control 2				
	Control 3				
	•••		•••	•••	

**Declaration:**

The GEM FLOW COUPLER Monitor is a device that is intended for use in an electromagnetic environment. The unit has met the performance requirements for Non-life supporting equipment pursuant to IEC 60601-1-2:2004 when subjected to an electrical field of 3V/m from 80 MHz with 80% AM at 1 kHz as well as an electrical field of 3 Vm from 800 MHz To 1 GHz with 100% amplitude modulation at 200 Hz square wave. The user of the Doppler system should assure it is being used in such an environment.

**Troubleshooting:**

There are no user serviceable components inside this device other than the batteries. Disassembly of the internal components of this unit may result in circuit damage.

**Troubleshooting Guide**

Symptoms	Possible Problem	Solution
No sound output	No power	Verify Monitor power is on
	No power	Check all connections: <ul style="list-style-type: none"> <li>• Probe Connector to External Lead</li> <li>• External Lead to Monitor</li> <li>• Monitor to AC Power Supply</li> <li>• AC Power Supply to Power Cord</li> <li>• Power Cord to outlet</li> </ul>
	Volume is too low	Adjust volume using Volume Increase switch
	Wrong channel is being used	Verify the correct channel is illuminated
	Batteries are dead	Replace batteries or use external power supply
	Monitor may be defective	Connect a different Monitor
	Monitor may be defective	Contact Synovis Micro Companies Alliance
Weak sound output	Weak batteries (low battery indicator is illuminated)	Replace batteries or use external power supply
	Volume is too low	Adjust volume using Volume Increase switch
	Monitor may be defective	Connect a different Monitor
	Monitor may be defective	Contact Synovis Micro Companies Alliance

**SERVICE:**

For Customer or Technical service, contact:

Phone: + 205.941.0111 or 1.800.510.3118 (U.S. only) • Fax: + 205.941.1522

Website: [synovismicro.com](http://synovismicro.com)

**STORAGE CONDITIONS:**

Recommended storage at 15 - 40°C (59 - 104°F)

**Accessories & Parts**

Item	Catalog Number
FLOW COUPLER Monitor	GEM1020M
AC Power Supply	GEM1007PS
Power Cord	GEM1009AC
Pole Mount	GEM1005-PM
Batteries	Standard
FLOW COUPLER Device 2.0	GEM2752-FC
FLOW COUPLER Device 2.5	GEM2753-FC
FLOW COUPLER Device 3.0	GEM2754-FC

**LIMITED WARRANTY**

The Synovis MCA FLOW COUPLER Monitor is warranted for one year from the date of shipment from Synovis MCA against defects in materials and workmanship. Defective Synovis MCA FLOW COUPLER Monitors will be repaired or replaced, at Synovis MCA's option, when returned prepaid to Synovis MCA within this year. The customer assumes full responsibility that this equipment meets the specifications, capabilities and other requirements of the customer. Synovis MCA makes no warranty of fitness for a particular purpose except as provided herein. The customer assumes full responsibility for the proper installation, operation and maintenance of this equipment as described in this manual as well as other instructions that may be provided by Synovis MCA. This warranty is void if the equipment has been mishandled, operated outside of its specified operating or environmental limits or otherwise subjected to improper or abnormal use.

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A Subsidiary of Synovis Life Technologies, Inc.