


## **INSTRUCTIONS FOR USE OF VISIBILITY** **BACKGROUND MATERIAL**

### **Description;**

Silicon Strip with 1mm Grid on one side and plain on reverse side, available in Blue, Green, and Yellow Colours. Packed in Boxes of 12 pcs.

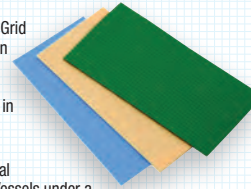

### **Application;**

For use in Micro Surgical Anastomosis of Micro Vessels under a Microscope to all the surgeon to better visualize the detail of the vessels being closed.

The Visibility Material can be used either with the Grid side up to give the Surgeon a scale of the vessel alternately the reverse Plain side can also be used.

The material has a more reflective surface eliminating distractions from the theatre lighting and has a non-tacky finish to aid mobilization of the Vessel.

### **Performance;**

The Silicon Material selected is suitable for use within the body.

A Barium mix is used to provide X-Ray Opaque quality to the Material.

The Material is Latex Free.

### **Precautions;**

Must only be used for its intended purpose - Visibility Material during Micro Surgery.

Not for implantation - must be removed from wound before closure of wound.

Not to be re-sterilized once opened.

Do not use if the packaging has been damaged or open.

### **Storage;**

These products should be stored under normal ambient conditions.

Life of the product limited to the date on the labeling.

### **Disposal;**

Can be disposed of without any special instructions and poses no danger if standard disposal regulations are observed.

**MERCIAN**  
Surgical Instruments of Excellence

Manufactured by Mercian Surgical Supply Co. Ltd., 10 Welverhampton Road, Warley, Birmingham B68 0LH, UK  
Distributed in the USA by Synovis Micro Companies Alliance, Inc., a division of Synovis Life Technologies,  
439 Industrial Lane, Birmingham, AL 35211 USA • 800-510-3518 • [www.synovismicro.com](http://www.synovismicro.com)