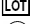
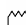
TITANIUM HEMOSTATIC CLIP**INSTRUCTIONS FOR USE****PLEASE READ ALL INFORMATION CAREFULLY**

Failure to properly follow the instructions may lead to serious surgical consequences.

IMPORTANT: This package insert is designed to provide instructions for the use of hemostatic clips. It is not a reference to surgical techniques.

	Read instructions for use		Reference
	Ethylene Oxide Sterilization		Lot Number
	Expiration date : YYYY-MM		Single use
	Manufacturing Date : YYYY-MM		Does not contain latex
	Check the package integrity before use. Sterility guaranteed unless package opened or damaged.		Do not re-sterilize
		Rx ONLY : Federal law (USA) restricts this device to sale by or on the order of a physician	

CONTENTS

This box contains according to the product reference in use:

GEM2431	30 cartridges of 6 MicroClips	Color Code: Orange
GEM1521	30 cartridges of 6 SuperFine MicroClips	Color Code: Blue

INDICATIONS

The titanium hemostatic clip has been specifically designed to securely grip the vessel or tissue to provide complete occlusion atraumatically. User shall select appropriate clip size according to procedure ensuring that all tissues are completely encompassed by clip.

Note

- This clip is not intended for Fallopian Tube Ligation.
- As a non-magnetic marking clip, it has low magnetic susceptibility and is safe for use in the MRI environment, with main magnetic fields up to 3 Tesla and a maximum main field gradient up to 500 Gauss per cm. Positional errors and artifacts may increase dramatically with increasing static magnetic field strength and/or increasing field of view size. Artifact size may vary greatly for different MRI pulse sequences. The titanium clip is biologically inert and can remain in vivo without sequela.

INSTRUCTIONS

- Inspect the integrity of the individual sterility protection (packaging) before using;
- Open the sterile packaging;
- Remove the protective strip from the adhesive base of the clip cartridge and press firmly on a stable surface;
- Select applier according to clip size (color coding);
- Hold applier at the box lock, or grasp one shank of the applier in the palm with forefinger at the box lock;
- Introduce the jaws of the applier vertically into the cartridge clip rack;
- Push the applier all the way into the cartridge rack and withdraw to remove the clip;
- Place the loaded clip and jaws of the applier around the vessel or tissue to be occluded;
- Firmly squeeze the handles together closing the clip. The clip closes tip to tip preventing vessel slippage before fully closing;
- Release the handle and remove applier.

WARNINGS**To prevent premature closure or release of the clip:**

- Never grasp the applier by both shanks when loading the clip;
- Do not squeeze applier handles before jaws are placed around vessel or tissue;
- Ensure applicators are fully closed to handle stop point when applying clip.

Inspect ligation site to ensure secure application and hemostasis.**To prevent damage to applier jaws:**

- Avoid introducing applier tips at an angle into the cartridge rack, this may misalign tips

Inspect applier and jaws before each use.**Use recommended applicators.****Open applier for thorough cleaning and sterilization.**

Federal (USA) law restricts this device to sale by or on the order of a physician.

STORAGE

Store the unopened sterile package in a cool dry area, with good ventilation away from areas of extreme temperature and humidity.

SHELF LIFE

Contents sterile if package is unopened, undamaged and within the validity date printed on the package.

THIS CLIP IS INTENDED FOR SINGLE USE AND CANNOT BE RESTERILIZED.

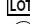
Check the packaging integrity before use. Sterility guaranteed unless package opened or damaged.

The manufacturer declines responsibility for problems resulting from improper use.

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