WARNINGS AND PRECAUTIONS

- Use with Peters Surgical Vitaltec® Hemostatic Clips.
- Appliers are provided NON-STERILE  and must be cleaned and sterilized according to these instructions prior to use.
- Ethylene oxide (EO), gas plasma and dry heat sterilization methods are not recommended for sterilization of clip appliers. Steam (moist heat) is the recommended method.
- Personal Protective Equipment (PPE) should be worn when handling or working with contaminated instruments.
- Caution should be exercised while handling, cleaning, or wiping instruments.
- Saline and cleaning/disinfection agents containing aldehyde, chloride, active chlorine, bromine, bromide, iodine or iodide are corrosive and should not be used.
- Do not allow biologic soil to dry on contaminated devices. All subsequent cleaning and sterilization steps are facilitated by not allowing blood, body fluids and tissue debris to dry on used instruments.
- Metal brushes and scouring pads must not be used during manual cleaning. These materials will damage the surface and finish of the clamps. Use only soft bristle brushes to aid with manual cleaning.
- When processing instruments do not place heavy devices on top of delicate instruments.
- Use of hard water should be avoided. Softened tap water may be used for most rinsing however purified water should be used for final rinsing to prevent mineral deposits.
- Oil's or silicone lubricants should not be used on clip appliers.
- Proper disposal of contaminated devices is required, including disposal of reusable clamps that are no longer acceptable for use.

CONTRAINDICATION

- Product is MRI non-compatible.

INSTRUCTIONS FOR USE AND INDICATIONS FOR USE

The Clip Appliers are designed to be used by medical professionals with Titanium Hemostatic Clips SLS-CLIP® VITALITEC and CLIP 9 VITALITEC® to perform the clip closure on the vessel.

PATIENT TARGET GROUP

Newborn to Geriatric

REPROCESSING

Reprocessing is in accordance with ISO 17664 and AAMI ST81. They apply to reusable clip appliers supplied by Vitaltec International and intended for reprocessing in a health care facility setting. Clip appliers may be safely and effectively reprocessed using the manual or combination manual/automated cleaning instructions and sterilization parameters provided in this document.

In countries where reprocessing requirements are more stringent than those provided in this document it is the responsibility of the user/processor to comply with those prevailing laws and ordinances. These reprocessing instructions have been validated as being capable of preparing reusable clip appliers for surgical use. It is the responsibility of the user/hospital/health care provider to ensure that reprocessing is performed using the appropriate equipment, materials and that personnel have been adequately trained in order to achieve the desired result; this normally requires that equipment and processes are validated and routinely monitored. Any deviation by the user/ hospital/health care provider from these instructions should be evaluated for effectiveness to avoid potential adverse consequences.

CLEANING OF CLIP APPLIER

Point of Use

- Remove excess biologic soil from the appliers with a disposable wipe. Place devices in a container of distilled water or cover with damp towels.
- If the appliers cannot be soaked or maintained damp then they should be cleaned as soon as possible (within 60 minutes is recommended) after use to minimize the potential for drying prior to cleaning.

Containment and Transportation

- Used appliers must be transported to the decontamination area for reprocessing in closed or covered containers to prevent unnecessary contamination risk.

Manual Cleaning

- Prepare a non-foaming, neutral pH enzymatic cleaner bath such as Enzol at the manufacturer's recommendation of 1 oz per gallon for processing clip appliers.
- Open the clip applier handle completely by disengaging the middle spring.
- Fully submerge the instruments and gently shake them to remove trapped bubbles. Actuate the instruments to ensure contact of the solution with all surfaces.
- Soak the appliers for at least 10 minutes. While soaking, scrub surfaces using a soft bristled brush until all visible soil has been removed. Actuate the clamp and pay particular attention to cleaning the boxlock.
- Remove the instruments from the enzyme solution and rinse in tap water for a minimum of one (1) minute. Open and close the clamp while rinsing. Make sure instrument surfaces are visibly clean and are free from stains and tissue.
- Dry instruments with a clean, lint-free cloth.
- Inspect each instrument for proper function and condition.

Automated Cleaning

- Rinse the appliers with water and while rinsing, scrub surfaces using a soft bristled brush until all visible soil has been removed. Actuate the instrument and pay particular attention to cleaning the boxlock.
- Place instruments in a suitable validated washer/disinfector. Follow the washer/disinfector manufacturer's instructions for loading the instruments for maximum cleaning exposure; e.g., open the applier, place concave instruments on their side or upside down, use baskets and trays designed for washers, place heavier instruments on the bottom of trays and baskets.
- The following **minimum** wash cycle parameters are recommended. The motor speed should be HIGH.

STAGE	RECIRCULATION TIME	TEMPERATURE	DETERGENT TYPE AND CONCENTRATION
PRE-WASH 1	2 MIN	COLD TAP WATER	N/A
ENZYME WASH	2 MIN	HOT TAP WATER	ENZOL 1 OZ/GAL
RINSE 1	15 SEC	43°C TAP WATER (SET POINT)	N/A
DRY TIME	7 MIN	90°C (SET POINT)	N/A

- If a lubrication cycle is available that applies a water-soluble lubricant such as Instrument Milk or equivalent, it is acceptable to use on the clip applier
- Make sure instrument surfaces are visibly clean and are free from stains and tissue.
- Dry instruments with a clean, lint-free cloth, if necessary.
- Inspect each instrument for proper function and condition.

Lubrication

- After cleaning and before sterilization, the appliers should be lubricated with a water-soluble lubricant such as Instrument Milk or equivalent. Always follow the lubricant manufacturer's instructions for dilution, shelf life and application method.
- The appliers must be terminally sterilized prior to use. See sterilization instructions below.

STERILIZATION OF CLIP APPLIER

- The clip appliers may be steam sterilized in instrument sets or individually in a separate peel pouch.
- Validated exposure times and temperatures to achieve a 10⁻⁶ sterility assurance level (SAL) are listed below.

Pre-vacuum Cycle

Preconditioning Pulses:	4
Temperature:	132°C / 270°F
Full Cycle Time:	4 Minutes
Dry Time:	30 Minutes

- A 30-minute cooling time is recommended after drying but longer times may be necessary because of load configuration and ambient temperature and humidity.
- Any other parameters used must be validated by the institution performing the cleaning and sterilization processes.

INSPECTION OF CLIP APPLIER

- The instrument should be inspected carefully prior to each use and during the cleaning procedure to identify any signs of unusual wear, cracking or corrosion. Microscopic cracks may be detected by the appearance of rust on the instrument. Should any irregularity be noted, the applier should not be used.

HOW SUPPLIED

The clip appliers are supplied non-sterile, one per package.

HANDLING

All surgical instruments should be handled with the greatest care, cleaned, treated, sterilized and stored. This is especially important for jaws, which are very sensitive areas.

STORAGE

Store in an area that is dry and clean, away from heat and chemical fumes.

CLINICAL BENEFITS

None

NOTICE

If any serious injury has occurred to the patient or user of this device, notify Vitaltec or appropriate authority with the details.

PRODUCT INFORMATION DISCLOSURE

Vitaltec excludes all warranties, whether expressed or implied by operation of law or otherwise, including, but not limited to: Any implied warranties of MERCHANTABILITY or FITNESS, since storage and handling of this device by the user, as well as factors relating to the patient, the diagnosis, treatment, surgical therapy and other matters beyond Vitaltec's control directly affect this device and the results obtained from its use. Vitaltec shall not be liable for INCIDENTAL or CONSEQUENTIAL LOSS, DAMAGE, or EXPENSE directly or indirectly arising from the use of this device. Vitaltec neither assumes, nor authorizes any other person to assume for it, any other or additional LIABILITY or RESPONSIBILITY in connection with this device.

INSTRUCTIONS FOR USE/GEBRAUCHSANWEISUNG/INSTRUCCIONES DE USO/MODE D'EMPOI/ISTRUZIONI PER L'USO/GEbruikSAANWIJZING/INSTRUÇÕES DE UTILIZAÇÃO GEM CLIP APPLIERS/GEM-CLIPAPPLIKATOR/APLICADORES DE CLIPS GEM/APPLICATEURS DE CLIPS GEM/APPLICATORE PER CLIP GEM/GEM-CLIPAPPLICATOR/APLICADORES/APLICADORES DE CLIPES GEM

	Rx Only	
Latex Free/Ohne Naturkautschuk hergestellt/No contiene látex/ Sans latex/Privo di lattice/ Latexvrij/Não contém látex	Prescription Only/ Verschreibungspflichtig/ De venta exclusiva con receta/ Sur ordonnance uniquement/ Solo su prescrizione/ Uitsluitend op voorschrift/ Sujeito a receita médica	Manufacturer/Hersteller/ Fabricante/Fabricant/ Fabricante/Fabrikant/ Fabricante
		
Unique Device Identifier/ Einmalige Produktkennung/ Identificador único del dispositivo/Identifiant unique de dispositif/Identificativo único del dispositivo/ Unieke apparaatidentificatie/ Identificador único do dispositivo	Lot Number/Chargennummer/ Número de lote/Número de lot/ Numero di lotto/Chargennummer/ Número do lote	Consult Instructions for Use/ Gebrauchsanweisung beachten/ Consultar las instrucciones de uso/Consultare le mode d'emploi/Consultare le istruzioni per l'uso/Gebruiksaanwijzing raadplegen/Consultar as instruções de utilização
		
Authorized Rep/ Bevoollmachtigter/Rep. autorizado/Rep. autorisê/ Mandatario/Gemachtigde vertegenwoordiger/Mandatário	Catalog Number/ Katalognummer/Referencia de catalogo/Número de referência/Numero di catalogo/ Catalogusnummer/Número de catálogo	Medical Device/Medizinprodukt/ Producto sanitario/Dispositif médical/Dispositivo medico/ Medisch hulpmiddel/Dispositivo médico
		
Importer/Importeur/Importador/ Importateur/Importatore/ Importeur/Importador	Distributor/Vertriebspartner/ Distribuidor/Distributeur/ Distributor/Distributeur/ Distribuidor	Non-sterile/Nicht steril/ No estéril/Non stérile/Non sterile/Niet-steriel/Não estéril

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